




RETROFIT INSTRUCTIONS


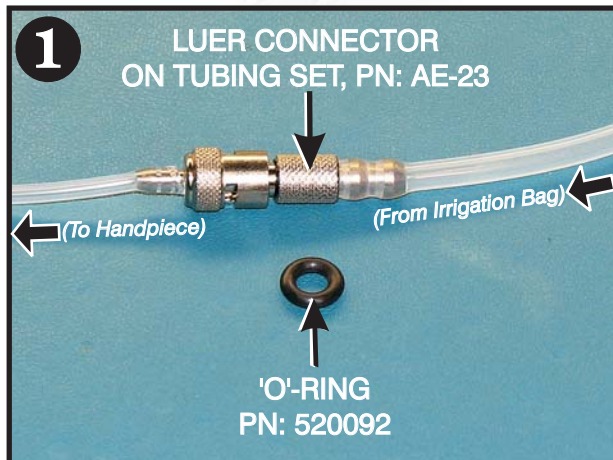


1 TOOLS REQUIRED(see Figure 1):

- Tubing Set, PN: AE-23
- 'O' Ring, PN: 520092

2 REMOVE IRRIGATION BAG TUBING FROM LUER CONNECTOR:

Remove irrigation bag Tubing from Luer Connector by grasping Connector and twisting and/or prying Tubing off barbed fitting (see Figures 2a & 2b).

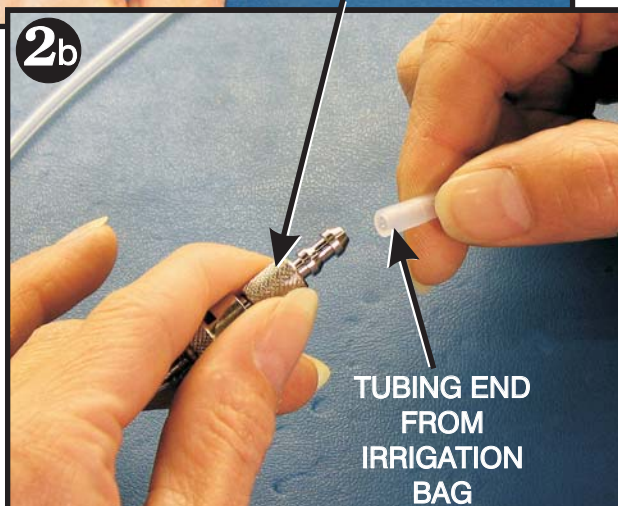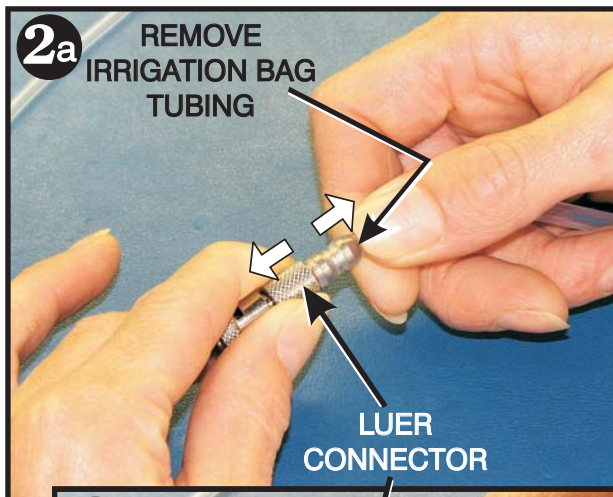




RETROFIT INSTRUCTIONS





3 INSTALL NEW 'O' RING ONTO TUBING END:

Position new 'O' Ring, PN: 520092, over end of irrigation bag Tubing and push Ring up tubing approximately 1-1/2" (see Figures 3a & 3b).

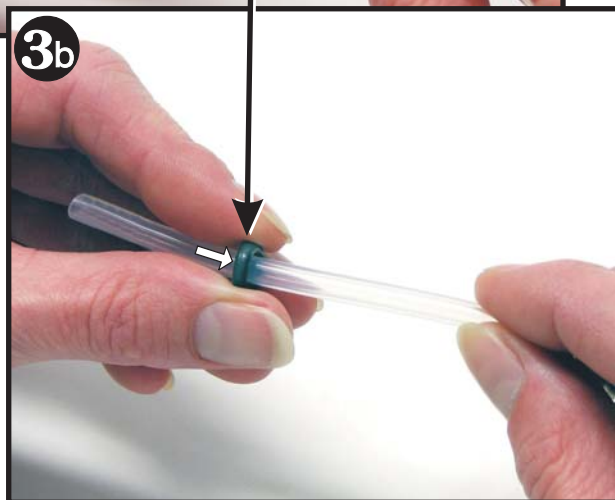

4 RECONNECT IRRIGATION BAG TUBING TO LUER CONNECTOR:

Reconnect the irrigation bag Tubing end to the barbed fitting on the Luer Connector. Grasp the end of the Tubing and push straight onto the fitting. Roll new 'O' Ring up tubing and onto Connector fitting until it seats flush against shank on Connector (see Figures 4a & 4b).

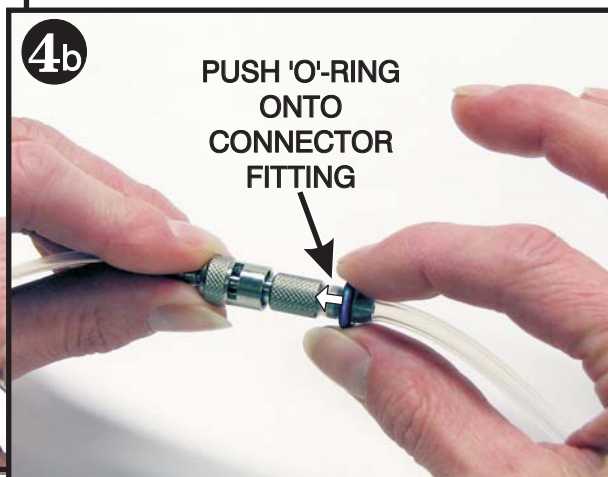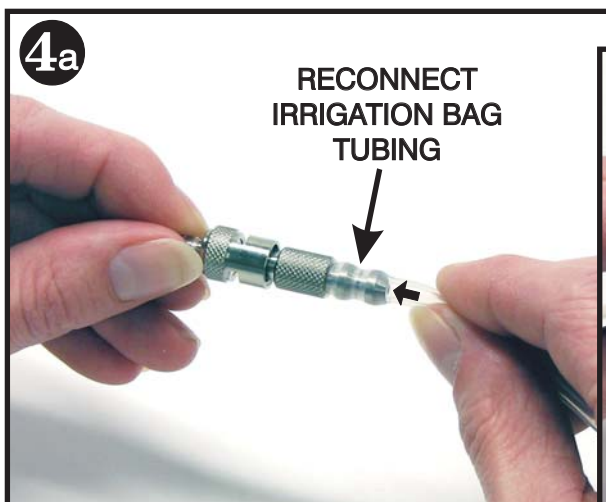



RETROFIT INSTRUCTIONS



- 5 COMPLETED RETROFIT:**
Sterilize the reassembled Tubing Set if necessary.

IMPORTANT: Sterilize the reassembled Tubing Set before each patient use. Sterilize at 132° C (270° F) for 10 minutes.

- 6** Install Tubing Set into the AEU-7000 System Pump Door per instructions provided in Operator's Manual. Grasp Luer Connector and gently pull outwards as shown in Figure 6a, then close and latch Pump Door. Slowly release tension on Luer Connector so that it seats against the outside of the case as shown in Figure 6b.



This completes the 'O' Ring retrofit instructions for the AEU-7000 System.

