

# AHP-85MB & AHP-85MB-C HANDPIECES

## IRRIGATION TUBING SETUP GUIDE

This guide provides step-by-step instructions for installing internal and external irrigation tubing onto the AHP-85MB and AHP-85MB-C Mont Blanc® contra angle handpieces.

(NOTE: AHP-85MB handpiece shown.)

**Step 1:** Remove Mont Blanc® handpiece (see Figure 1, Item #1) from box and install onto dental motor (consult motor operator's instructions). Insert rotating instrument into head of handpiece until "click" is heard.




**Step 2:** Preassemble Items #2, #3, #4, and #5 into a subassembly, as shown in Figure 2.

**Step 3:** Mount subassembly from Step 2 onto handpiece/motor, as shown in Figure 3.

**Step 4:** Connect tubing components with tubing from irrigation pump, as shown in Figure 3.

**NOTE:** When tubing is set up for simultaneous internal/external irrigation, the irrigation fluid will favor the line of least resistance, so the bulk of the fluid will flow to the external port.

**NOTE:** To set up irrigation for either internal only, or external only, connect the tubing w/small ID directly to the desired internal or external port — do not use the 'Y' connector.

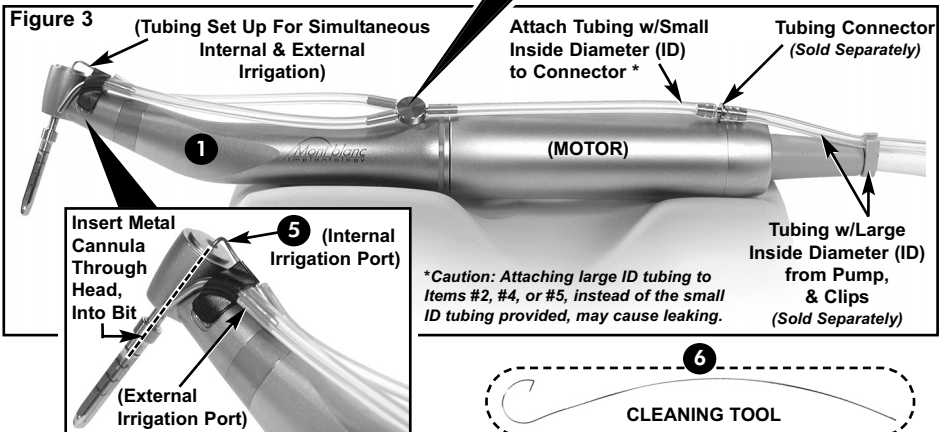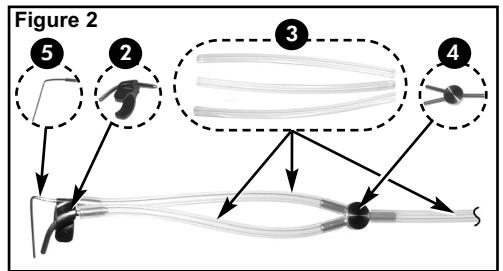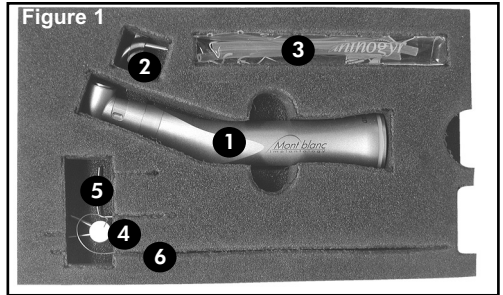

©ASEPTICO®

8333 216th Street SE  
Woodinville, WA 98072

Phone: 425-487-3157 • www.aseptico.com

P/N: 420744 Rev. A ECO 11713 01/2008