



Box 1548 Woodinville, WA 98072
425-487-3157 • email - info@aseptico.com

INSTRUCTIONS FOR USE

ARU-01A - Portable X-Ray Stand Extension Arm

1. Fully extend front two legs of tripod stand.
2. Extend rear leg approximately half way out.
3. Attach Extension Arm onto the adjustable center post.
4. With assistant, mount X-Ray as shown, use washer and wing nut.
5. Take care in making sure all connections are tightened, both on tripod stand and extension arm.
6. Carefully check balance of assembled system, ensuring the unit will not easily tip over. Adjust rear leg on tripod stand as necessary.
7. Height adjustment to X-Ray head can be achieved by raising and lowering height adjustment on tripod stand.

